




i-gel[®] O₂ Resus Pack

Four things in one pack, one less thing to think about



Airway Management ▪ Airway Devices

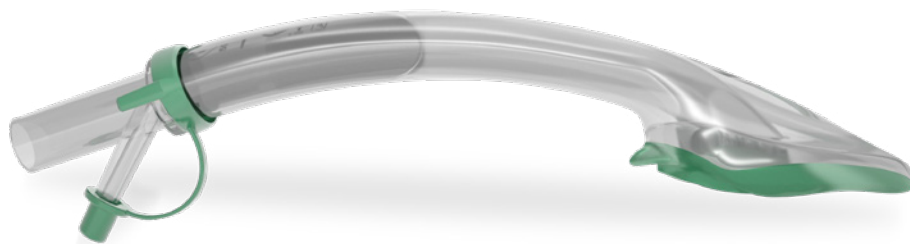

In emergency medicine, you need equipment that's easy, rapid and reliable to use. The i-gel® O₂ Resus Pack contains everything you need to prepare, insert and secure the i-gel® O₂ quickly and efficiently: an i-gel® O₂ supraglottic airway, a sachet of lubricant, airway support strap and a suction tube.

The i-gel® O₂ incorporates a supplementary oxygen port and an integral hook ring for securing of the airway support strap.

The i-gel® O₂ has been designed to facilitate ventilation as part of standard resuscitation protocols, such as those designated by the European Resuscitation Council (ERC) and the American Heart Association (AHA). However, the i-gel® O₂ incorporates a supplementary oxygen port, so can also be used for the delivery of passive oxygenation, or Passive Airway Management (PAM™), as part of an appropriate CardioCerebral Resuscitation (CCR) protocol.



Airway innovation since 2007

i-gel® is the innovative second generation supraglottic airway device from Intersurgical launched in 2007. The first major development since the laryngeal mask airway, the i-gel® changed the face of airway management and is now widely used in anaesthesia and resuscitation around the world.



The non-inflatable cuff

The i-gel® gets its name from the innovative soft, gel-like material from which it is made. It is the innovative application of this material that has enabled the development of a unique non-inflatable cuff. This means there is

no need for cuff deflation prior to insertion and no cuff inflation after placement to secure a seal, shortening and simplifying the preparation and insertion procedure.

Insertion

The i-gel® O₂ is incredibly easy to use. Insertion is rapid and can normally be achieved in less than 5 seconds. The simplicity of i-gel® O₂ means the insertion technique is instinctive, reliable and does not require or benefit from the insertion of fingers or thumbs into the patient's mouth.

Fixation

The i-gel® O₂ Resus Pack includes a specially designed airway support strap for securing the i-gel® O₂ in position. Fixation is easily achieved by sliding the strap under the patient's neck and securing it in place to the hook ring on the collar of the i-gel® O₂. This makes it ideal for use where adhesive tape is unsuitable, such as on patients with facial hair.



i-gel® O₂ Resus Pack

15mm connector

Reliable connection to any standard catheter mount or connection

Proximal end of gastric channel

Colour coded hook ring

To secure the i-gel® O₂ in position with the airway support strap

Clearly displayed product information

For quick easy reference. Includes confirmation of size and weight guidance

Position guide

Supplementary oxygen port

For the administration of passive oxygenation as a component of cardiocerebral resuscitation (CCR)

Gastric channel

Enhances patient safety by providing a mechanism for the management of regurgitant fluid

Integral bite block

Reduces the possibility of airway channel occlusion

Buccal cavity stabiliser

Aids insertion and eliminates the potential for rotation

Epiglottic rest

Reduces the possibility of epiglottis 'down folding' and airway obstruction

Non-inflatable cuff

Eliminates the need for cuff inflation after insertion, allowing easy and rapid insertion

Distal end of gastric channel



The i-gel® O₂ Resus Pack is available in 3 adult sizes and includes:

- i-gel® O₂ supraglottic airway
- Sachet of lubricant – for quick and easy lubrication of the i-gel® O₂ prior to insertion
- Airway support strap – to secure the i-gel® O₂ in position
- 12 FG suction tube – for insertion through the gastric channel to empty fluid from the stomach.



Ordering information

The i-gel® O₂ Resus Pack contains an i-gel® O₂, a sachet of lubricant, airway support strap and suction tube, all packaged in a specially designed clear, sterile, rigid pack. This unique design ensures protection of the contents, both in transit and in storage. The i-gel® O₂ has a colour coded hook ring to allow quick and easy identification of size. The i-gel® O₂ Resus Pack is available in three adult sizes.



Code	Description	Weight	Box Qty.
8705000	i-gel® O ₂ Resus Pack, large adult – includes a size 5 i-gel® O ₂ with orange hook ring, sachet of lubricant, airway support strap and a 12FG suction tube	90+kg	6 
8704000	i-gel® O ₂ Resus Pack, medium adult – includes a size 4 i-gel® O ₂ with green hook ring, sachet of lubricant, airway support strap and a 12FG suction tube	50–90kg	6 
8703000	i-gel® O ₂ Resus Pack, small adult – includes a size 3 i-gel® O ₂ with yellow hook ring, sachet of lubricant, airway support strap and a 12FG suction tube	30–60kg	6 

 Sterile

Please note: Suction tube contains PVC.



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK

T: +44 (0)118 965 6300 F: +44 (0)118 965 6356 info@intersurgical.com www.intersurgical.com



The manufacturer Intersurgical Ltd is certified to ISO 9001:2015, ISO 13485:2016 and ISO 14001:2015

Please think before you print
Save energy and paper.
If you must print this information sheet please print it double sided.

IS6.6 • Issue 9 09.21